

BEFORE AND AFTER

The technology to **Diodo** allows outstanding results



EPIL
EVOLUTION
808

Castelfidardo (AN) Italy - Via E.Berlinguer, 11
www.epil-evolution808.it - info@epil-evolution808.it
Simone cell. 338.5353693 - Fabrizio cell. 335.1337627

AUTHORIZED DEALER

EPIL
EVOLUTION
808



FOR PERMANENT
HAIR REMOVAL

DIODO
LASER
808

NEW GENERATION

WHAT IS THE DIODE LASER?

The Diode laser 808nm is used for permanent hair removal of the entire body, for men and women, for the whole year and for every phototype.

Our laser equipment, **Made in Italy**, is specifically designed and built for permanent hair removal and in the absolute compliance with international safety standards and regulations.

As well known, the physical principle, which the Diode laser is based on, is the selective photothermolysis. By this process, pigment cells continuously gather in the hair follicle and along the hair shaft. The laser's ray aims selectively at the pigment cell without damaging adjacent tissues. Pigment cells, by absorbing the heat produced by photo-thermic energy's distribution, undergo an increase of temperature that leads to the hair follicle's devitalization. This process causes the hair follicle to fall and to the following inhibition of regrowth.

The treatment with the Diode laser, compared to the most used technologies for hair removal, has proven to be more efficient, faster and safer.



EPIL
EVOLUTION
808

● FOR WHOM AND WHERE IS RECOMMENDED?

It is excellent both for men and women with every skin and hair color conditions and for every season of the year.

● WHEN CAN IT NOT BE USED?

It can't be used on skin wounds, dermatological pathologies, minors, people who use hormonal and cortisone medicines and on white hair.

● HOW MANY TREATMENTS ARE NEEDED?

The number of treatments has to be decided together evaluating the starting conditions of skin, hair and previous depilatory habits. The minimum number of treatments may vary between 5 and 10.

● IS IT PAINFUL?

Thanks to the double air and water cooling system and to the speed of execution with the multi-impulse mode, it is, by far, the lesser discomforting technique for every kind of skin.

You will feel the frozen handle cooling your skin and in a few minutes your inferior limb will be completely treated, for example.

● HOW MUCH DOES IT COST?

It depends on the area to be treated and on the number of necessary applications.

